DEUTSCH

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ENGLISH

Important safety information – retain for later use.

In order to make optimum use of all the merits of the clinical thermometer, you should carefully read the operating instructions before using it, retain the instructions for later use and also keep them accessible for other users.

- The thermometer is intended to measure the temperature of the human body.
- Children should not use the thermometer without supervision.
- Before each use, check the thermometer for signs of damage or wear. Damaged or worn thermometers should not be used.
- Always wait for the acoustic signal which indicates the minimum time required for measurement. Discuss the temperature with your family physician.
- The thermometer contains sensitive electronic parts. Therefore, protect it from blows, bending, high temperature or direct exposure to sunlight.
- Using the thermometer in strong electromagnetic fields such as next to a mobile phone, can cause a malfunction.
- When the thermometer is switched on, it carries out a self-test. Therefore the accuracy of measurement need not be tested at regular intervals.
- If the temperature is very abnormal, please contact your family doctor promptly.
- This unit is in line with European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit. For more details, please contact customer service at the address indicated.

Measuring methods:

Temperature measurement in the anus (rectal)

This method of measurement is most reliable and most accurate. Guide the tip of the thermometer carefully 2 to 3 cm into the anus.

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ampit, the thermometer should be left in the ampit for the recommended time of measurement, in spite of the acoustic signal. If temperatures over 37.5 °C are measured, a fever alarm sounds (short bleeping tones, 4 seconds long). If temperatures higher than 44 °C are measured, the symbol for »H °C« will appear. To prolong the life of the battery, please switch off the thermometer by briefly pressing the on/off key after the measurement. Otherwise the thermometer will automatically switch off after 10 minutes.

Memory

The last measured value will be stored automatically. In order to display the stored temperature, press the on/off key for longer than 3 seconds. Next to the measured value the storage symbol will appear, e.g. 37.0 °CM. When the ON/OFF button is released, the measured value is deleted, the reference value of 36.5 °C reappears briefly and the thermometer is again ready for use.

Changing the battery

The clinical thermometer contains a long-life alkali-manganese battery (LR 41, 1.55 V), which will last for approx. 2 years. The battery has to be replaced when "T" appears in the bottom right corner of the display. Press down the battery cover at the end of the device with your fingers and then pull it firmly back. Remove the old battery and replace it with a battery of the same type. The + sign on the battery must point upwards. Then put the battery cover back onto the case. Used batteries do not belong in the regular domestic refuse. You are legally obliged to dispose of the batteries. Dispose of them through your local dealer for electric goods or your local depot for special waste. You are legally obliged to dispose of the batteries. Note: Batteries containing pollutant substances are marked as follows: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury. These batteries contain lead (Pb) and mercury (Hg).

Cleaning/Disinfection

The thermometer is 100% water-proof. For the purpose of cleaning or disinfection, it can be dipped into water or disinfectant without any difficulties. Under no circumstances should the thermometer be boiled! Use one of the following disinfectants in accordance with the manufacturer's

The thermometer is 100% water-proof. For the purpose of cleaning or disinfection, it can be dipped into water or disinfectant without any difficulties. Under no circumstances should the thermometer be boiled! Use one of the following disinfectants in accordance with the manufacturer's

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