- | | |
|--|--|
| (D) Glaswaage
Gebrauchsanleitung | (I) Bilancia in vetro
Istruzioni per l'uso |
| (GB) Glass scale
Instruction for Use | (TR) Cam Terazı
Kullanma Talimatı |
| (F) Pèse-personne en verre
Mode d'emploi | (RUS) Стеклянные весы
Инструкция по
применению |
| (E) Báscula de vidrio para
personas
Instrucciones para el uso | (PL) Waga szklana
Instrukcja obsługi |

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Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,
Your Beurer Team

1 Getting to know your instrument

Function of the unit

These digital glass scale allows you to weigh yourself and to calculate your personal body mass index (BMI).

The scale is intended for individual, household use.

The scale is equipped with the following functions, which can be used by up to ten different people:

- Measurement of your body weight
- Calculation of your body mass index (BMI)
- Coloured interpretation of your BMI for simple rating.

The memory function lets up to ten people access up to ten previously measured weight values as well as the corresponding BMI values.

2 Signs and symbols

The following symbols appear in these instructions.

**Warning**

Warning instruction indicating a risk of injury or damage to health.

**Important**

Safety note indicating possible damage to the unit/accessory.

**Note**

Note on important information.

3 Safety notes

Safety notes**Warning:**

- Do not stand on the outer edge to one side of the scale otherwise it may tip!
- Keep children away from packaging materials (risk of suffocation).
- The surface of the scale is slippery when damp or wet.

Notes on handling batteries

Warning:

- Swallowing batteries can be extremely dangerous. Keep the batteries and scale out of the reach of small children. Should a child swallow a battery, seek medical assistance immediately.
 - Batteries must not be recharged, taken apart, thrown into an open fire or short circuited.
 - Batteries can contain toxins that are harmful to health and the environment. Always dispose of batteries in accordance with applicable legal regulations. Do not dispose of batteries with the normal household waste.
-

Warning:

- Replace weak batteries before they discharge completely.
 - Always replace all batteries at the same time and use batteries of the same type.
 - Leaking batteries may damage the device. If you do not intend to use the device for longer periods, remove the batteries from the battery compartment.
 - If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
-

General notes


- The unit is for personal use only and is not intended for medical or commercial applications.
- Please note that technically related measuring tolerances are possible, as the scale is not calibrated for professional medical applications.
- The scale can accept a maximum load of 150 kg (330 lb, 23 st). The weight measurement is displayed in increments of 100 g (0.2 lb).
- Before submitting any complaint, first check the batteries and replace them if necessary.
- Repairs must only be carried out by Customer Services or authorised suppliers.
- Should you have any questions concerning the use of our devices, please contact your dealer or customer service.
- All scales comply with EC Directive 2004/108/EC.

4 Unit description

Overview

- 1 Display
- 2 Up button 
- 3 USER button
- 4 Down button 
- 5 SET button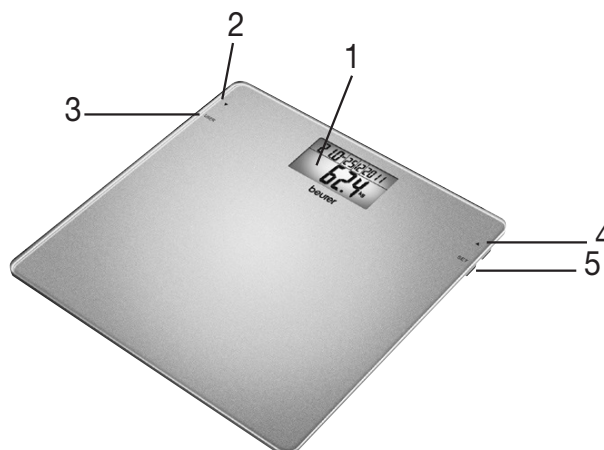

5 Initial use

Inserting the batteries (3 x 1,5 V AAA Micro)

Please remove the battery insulating strips and/or position the batteries correctly with regards to the polarity displayed in the battery compartment. If the scale fails to operate, remove the batteries completely and reinsert them.

Changing the weight unit

The scale shows the values in kg. You can set the device to measure in kilograms ('kg'), pounds ('lb') and stones ('st') using the button on the back of the device.

Setting up the scale

Set up the scale on a level and solid surface. A solid surface is a prerequisite for a correct measurement.

6 Information

The measuring principle

The BMI is calculated using the height value entered and the measured weight value.


General tips

- If possible, always weigh yourself at the same time of day (preferably in the morning), after using the toilet, on an empty stomach and unclothed, in order to achieve comparable results.

7 Switch on the scale


Tap your foot quickly, forcefully and audibly on the scale to activate the vibration sensor.

'8888' and the date appear as a self test until '0.0' is displayed. If necessary, change the location where the user data is saved.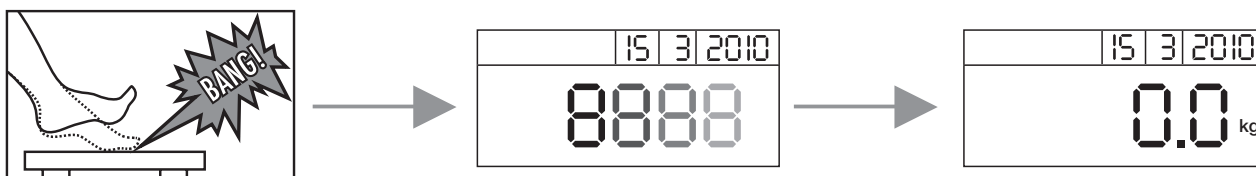

8 Settings

Set time and date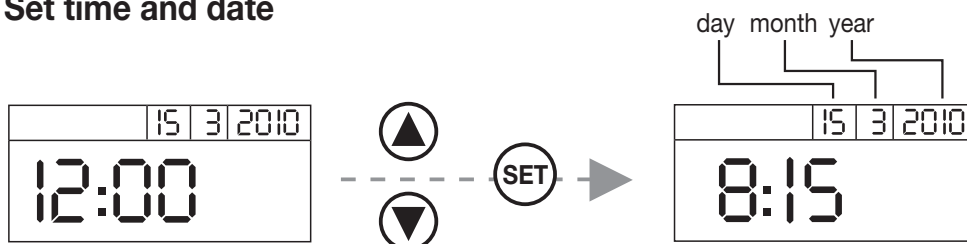

The date and time can only be changed when the batteries are taken out and replaced.

To set the current date and time, you must proceed in several stages as follows:

day – month – year – hour – minute.

The setting to be changed will flash each time.

- To change a setting, press the buttons ▽ and △ .
- To save the setting and move onto the next one, or to close the setting, press the SET button. The scale will switch off automatically after this procedure.

The scale will also switch off if no buttons are pressed for more than 15 seconds.


Setting user data

The scale comes equipped with 10 locations for storing user data and space for more than 10 measurement values per user. You and, for example, the members of your family can save and later access the following personal settings in the storage locations:

Sequence	User data	Parameters
1	Memory position	1 to 10
2	Body height	100 to 220 cm (3'-03" to 7'-03")

Switch on the scale (see 7. Switch on the scale). When “0.0 kg” is displayed:

- Press the 'USER' button until you have reached the desired memory space. After the display shows the preset body height “0.0 kg” will appear.
- To change the body height press “SET”.
- Press the ▽ and △ buttons to set your height. Keep button pressed for quicker run.
- Finish entering data by pressing the SET button.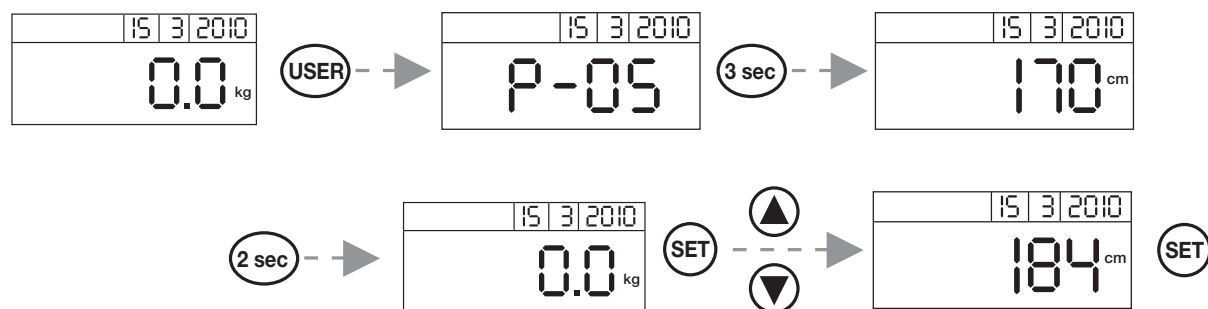


i Please note: if you change the height value at a later date, all measurement data saved in your storage location up until that point will be deleted.

9 Operation

Weighing yourself and calculating BMI

To weigh yourself and to calculate your personal body mass index:

- Switch on the scale (see 7. Switch on the scale).
- Then select the storage location where your settings have been saved using the 'USER' button. Next, your saved height value will be displayed. Wait until “0.0 kg” is displayed again. You can now use the scale. Stand still whilst weighing yourself.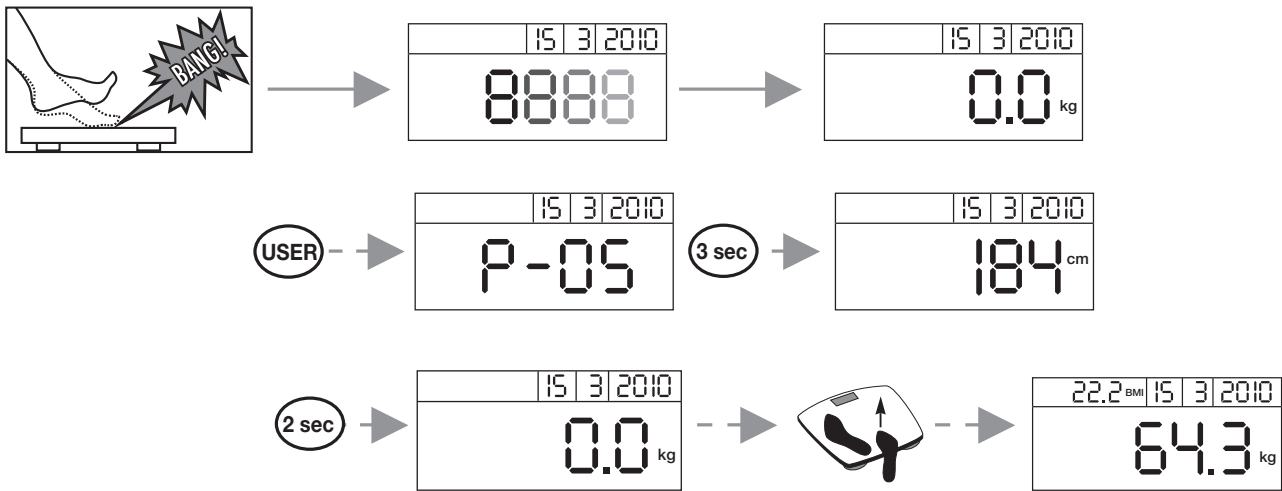

The date, weight and BMI are each displayed.

The display changes colour depending on the BMI (see page 10, Evaluate results).


One value will be saved per day, per person. If you take several measurements per day, the most recent value will be saved. The data saved previously on that day up will be overwritten.

The scale switches itself off after approx. 10 seconds.

Accessing saved measurements

To access the saved measurements of the currently selected user:

- Switch on the scale (see 7. Switch on the scale). '0.0 kg' is displayed after the self-test display.
- Select the memory space by pressing the 'USER' button several times, if necessary. The height of the selected user and then his/her last saved result is displayed again if you do not press any buttons for three seconds after that '0.0 kg' will appear.
- Then browse through the memory spaces for the saved values one by one using the ∇ or \triangle buttons.
- The date, weight and BMI are each displayed. The display changes colour depending on the BMI (see 10, Evaluation of results).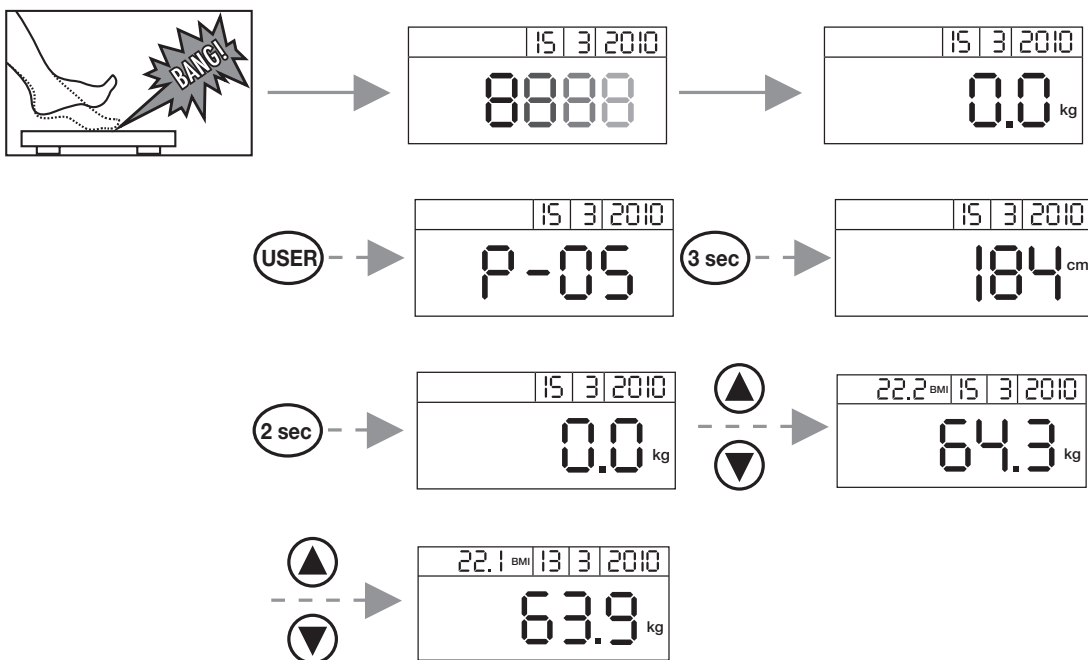

10 Evaluation of results

Body mass index (BMI)

The body mass index (BMI) is a number that is often called upon to evaluate body weight. The number is calculated from body weight and height, the formula is – body mass index = body weight : height². The measurement unit for BMI is [kg/m²]. According to the BMI, weight is classified for adults (20 years and over) using the following values (source: WHO):

Display	Category		BMI
Pink	Underweight	Severely underweight	< 16
		Moderately underweight	16-16,9
		Slightly underweight	17-18,4
Green	Normal weight		18,5-25
Yellow	Overweight	Pre-obese	25,1-29,9
Red	Obese (overweight)	Class I obese	30-34,9
		Class II obese	35-39,9
		Class III obese	≥ 40

Results in relation to time

 **Remember that only long-term trends are important. Short-term fluctuations in weight over a few days are usually the result of a loss of fluid.**

11 Replacing batteries

The batteries in the scale must be replaced if the message “Lo” appears in the display. If the batteries are too flat, then the scale will switch off automatically.

Note:

- When changing batteries, always use batteries of the same type, brand and capacity. Always replace all batteries simultaneously.
- Do not use rechargeable batteries.
- Use batteries that are free of heavy metals.

12 Storage and maintenance

The precision of the measurements and service life of the device depends on its careful handling:

Important:


- The unit should be cleaned occasionally. Use a slightly damp cloth to clean the device. You may also use a small quantity of cleaning agent if desired. Do not use abrasive detergents and never immerse the unit in water.
- Protect the scale from knocks, damp, dust, chemicals, drastic changes in temperature, electromagnetic fields and nearby sources of heat (ovens, heaters). Do not press the buttons violently or with pointed objects.

13 Disposal

Standard and rechargeable batteries should not be disposed of separately from the household waste. As a consumer, you are legally obliged to return used batteries for proper disposal. You can hand in your used batteries at public collection points in your district or sales outlets where batteries of this type are sold.

Note: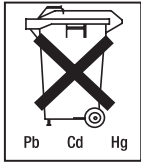

The codes below are printed on batteries containing harmful substances:
 Pb = Battery contains lead,
 Cd = Battery contains cadmium,
 Hg = Battery contains mercury.

For environmental reasons, do not dispose of the scale with the batteries in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Observe the local regulations for material disposal.

Dispose of the unit in accordance with EC Directive **2002/96/EC – WEEE** (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

14 What if there are problems?

If the scale detects an error when measuring, the following is displayed.

Display	Cause	Remedy
OL	The maximum load capacity of 150 kg was exceeded.	Do not load the unit with over 150 kg.
Lo	The batteries are almost empty.	Replace the batteries.
no	Accessing a user storage location in which no values have been saved.	The scale will switch off automatically after a short while; switch the scale on again and proceed with the measurements.
Err	BMI is outside the measurable range.	Switch the scale on again and repeat the measurements.

No measurement possible

Possible errors	Remedy
If you stand on the scale to measure your values before the display shows “0.0 kg”, the scale will not function correctly.	Correctly activate the scale (wait for “0.0 kg” to be displayed) and repeat the measurement.
The batteries in the scale are empty.	Replace the batteries.

Incorrect weight measurement

Possible errors	Remedy
Scale is on carpeting.	Place the scale on a level and solid surface.
Incorrect zero point of scale.	Place a scale on a secure, flat surface. Switch on the scale and wait until the scale automatically switches off again (do not step onto the scale before then). Then switch on the scale again and repeat the measurement.

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