- D** Installationsanleitung
- GB** Installation instructions
- F** Instructions d'installation
- E** Guía de instalación
- I** Istruzioni per l'installazione
- TR** Kurulum kılavuzu
- RUS** Руководство по установке
- PL** Instrukcja instalacji

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English

1 System requirements

- Windows XP or Vista operating system, and the use of Microsoft Excel as analysis software, are system requirements.

2 Installing software

- Load the CD.
- Open the workstation with Explorer and then the CD drive.
- Copy all files to the clipboard.
- Create a new directory (folder) on your hard drive, for example by clicking on C:\Programs "File" → "New" → "Folder" and give the folder a name, e.g. "BF100".
- Paste all the CD data from the clipboard to this directory.
- Create a desktop shortcut for the "BF100.exe" file by right clicking the file, selecting "Send to" and then selecting "Desktop" (create shortcut).

3 Using the software

The control unit must be in standby mode (date and time displayed).

- Double click the "BF100.exe" file in the newly created directory, or the shortcut which you created on your desktop. The PC software will start.
- Connect the unit to the PC with the USB cable. Insert the flat USB connector of the connection cable into a USB port on your PC. Plug the smaller mini

USB plug into the USB ports on the right-hand side of the control unit.

- "CONNECTED VIA_USB" appears on the unit's display (or in another language, if you selected one).

- Within 30 seconds of connecting the USB cable, click on “UPLOAD” to start the transfer of data from the control unit to the PC. The data is saved in the user database.
- The message “UPLOAD IS COMPLETE” appears when the data has been fully transferred. Confirm by clicking on “OK”.
- When the message “OPEN DATABASE?” appears, click on “OK” to view the data.
- Click on “EXIT” to end the PC program.

 You can also open the data by opening the “UserDatabase.xls” file in a spreadsheet program.

4 Notes

- The control unit data remains saved in the control unit and is not deleted during transfer.
- After saving a new measurement, you can transfer the data to your PC at any time. The more recent data records (measurements) are each automatically added to the database.
- If you want to start again with an empty database, delete the “UserDatabase.xls” file in your directory or rename the file. If required for reasons of security, copy the original file to another folder.
- Do not amend the files which have been installed. In particular, the “DefaultDatabase.xls” file must not be amended. Doing so could cause data problems.
- Do not open the XLS file if a USB device is currently connected or data is being transferred.
- If the error message “NO USB CONNECTION” appears, check the USB connection between the unit and the PC. Disconnect the USB connection for approx. 10 seconds and reconnect the control unit to the PC.

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