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Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,
Your Beurer Team

1. Interesting facts

Weighing correctly

If possible, always weigh yourself at the same time of day (ideally in the morning), after going to the toilet, on an empty stomach and without clothes to achieve comparable results.

2. Important notes – keep for later use!

■ Caution – For your safety

- Attention! Do not step onto the scale with wet feet or when the scale's surface is damp -danger of slipping!
- You should protect the scale against impacts, moisture, dust, chemicals, major temperature fluctuations and keep it away from heat sources (ovens, radiators).
- Repairs may only be performed by Beurer Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making a complaint.
- All scales comply with EC Directive 2004/108 + supplements. Should you still have questions regarding the use of our products, please contact your dealer or Beurer Customer Service.
- The scale should occasionally be cleaned with a damp cloth. Do not use aggressive cleaning agents and never submerge the scale.
- The device is only for private use and is not intended for medical or commercial purposes.
- On delivery the scale is set to the unit "kg". A switch located on the back of the scale enables you to change to "pound" and "stones" (lb, St).
- Please dispose of the appliance in accordance with the Directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE). In case of queries, please contact the municipal authorities responsible for waste disposal in your area.

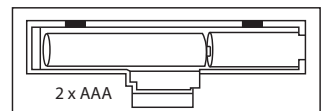
3. Use

This scale is used to determine the weight of human beings. The scale has a weighing range up to 150 kg (330 lbs). If the scale is loaded with a higher weight (>150 kg), "- - -" appears in the display.


3.1 Batteries

If present, pull the battery insulating strip off the battery compartment cover or remove the battery's protective film and insert the battery according to the polarity. If the scale fails to operate, remove the battery completely and insert it again. Your scale is equipped with a "replace battery" indicator. When the scale is

operated with a battery which is too weak, "Lo" appears in the display field and the scale automatically switches off. The battery must be replaced in this case (2 x 1,5 V, AAA, Mikro). The used, completely drained standard and rechargeable batteries must be disposed of in specially marked

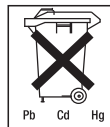collection containers, at toxic waste collection points or electrical product retailers. You are legally obligated to dispose of the batteries.

Note: Batteries containing pollutants are marked with the following symbols: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

3.2 Measuring weight

Place the scale on a firm, flat surface (not on carpet); a firm floor covering is a basic requirement for correct weight measurement.

Fig. 1

Stand still on the scale with your weight distributed evenly between both legs. „0.0“ appears in the display (Fig. 1). The scale immediately begins to measure your weight. After the display flashes twice, the final measurement result appears on the display (Fig. 2). The scale switches off approx. 10 seconds after you step down from it. The measuring result remains visible during this time.

Fig. 2

If the scale detects an error during weighing, „ - - - - “ appears in the display (Fig. 3).

Fig. 3

Note: When using for the first time or if the location of the scale has been changed, tap the standing surface of the scale without standing on the scale. After the display switches off the normal measurement can be carried out.

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