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## ENGLISH

### Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,  
Your Beurer Team

## 1. Interesting facts

### Weighing correctly

If possible, always weigh yourself at the same time of day (ideally in the morning), after going to the toilet, on an empty stomach and without clothes to achieve comparable results.

## 2. Important notes – keep for later use!

### Caution – For your safety

- Attention! Do not step onto the scale with wet feet or when the scale's surface is damp -danger of slipping!
- You should protect the scale against impacts, moisture, dust, chemicals, major temperature fluctuations and keep it away from heat sources (ovens, radiators).
- Repairs may only be performed by Beurer Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making a complaint.
- All scales comply with EC Directive 2004/108 + supplements. Should you still have questions regarding the use of our products, please contact your dealer or Beurer Customer Service.
- The scale should occasionally be cleaned with a damp cloth. Do not use aggressive cleaning agents and never submerge the scale.
- The device is only for private use and is not intended for medical or commercial purposes.
- On delivery the scale is set to the unit „kg“. A switch located on the back of the scale enables you to change to „pounds“ (lb).

– Please dispose of the appliance in accordance with the Directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE). In case of queries, please contact the municipal authorities responsible for waste disposal in your area.

### 3. Voice function

You can set the voice function of the scale in the switched off state. The device offers 2 languages (German (D) and English (GB)). The device is set to German on delivery. To change the language selection, set the button on the back of the scale to “GB” for English.

Volume adjustment: You can set or switch off the language function by setting the button above the loudspeaker symbol at the back of the scale as follows:

- **Left:** language function is switched off
- **Middle:** quiet
- **Right:** loud

### 4. Use

This scale is used to determine the weight of human beings. The scale has a weighing range up to 396 lbs (180 kg). If the scale is loaded with a higher weight (>180 kg), “Err” appears in the display, as well as the following voice output:

Display	Language D	Language GB
Err	Fehler	Error

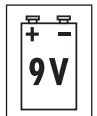
#### 4.1 Start-up

If present, pull the battery insulating strip off the battery compartment cover or remove the battery’s protective film and insert the battery according to the polarity. Ensure that you position the tab beneath the battery so the battery is easier to replace later. If the scale fails to operate, remove the battery completely and insert it again. Your scale is equipped with a „replace battery“ indicator. When the scale is operated with a battery which is too weak, „Lo“ appears in the display field and the scale automatically switches off.

Display	Language D	Language GB
Lo	Wenig Batterie	Low battery

The battery must be replaced in this case (1 x 9V battery). The used, completely drained standard and rechargeable batteries must be disposed of in specially marked collection containers, at toxic waste collection points or electrical product retailers. You are legally obligated to dispose of the batteries.

**Note:** Batteries containing pollutants are marked with the following symbols:

- Pb = Battery contains lead,
- Cd = Battery contains cadmium,
- Hg = Battery contains mercury.

#### 4.2 Measuring weight

Place the scale on a firm, flat surface (not on carpet); a firm floor covering is a basic requirement for correct weight measurement.

Quickly and firmly tap the lower part of the standing surface of your scale with your foot. The entire display (Fig. 1) appears up to „0.0“ (Fig. 2) as a self-test.

Fig. 1

Fig. 2

Display	Language D	Language GB
8888	Bitte warten	Please wait

Now the scale is ready to measure your weight.

Display	Language D	Language GB
0.0	Bitte hinauf steigen	Please step on the scale

Now step onto the scale. Stand still on the scale with your weight distributed evenly between both legs. The scale immediately begins to measure your weight. After flashing twice the final measurement result appears on the display. (Fig. 3) There is also an acoustic output. The scale switches off 10 seconds after you step down from it. The measuring result remains visible during this time.

Fig. 3

Fig. 4

If the scale detects an error during weighing, „Err“ appears in the display (Fig. 4).

Display	Language D	Language GB
Err	Fehler	Error

Make sure that you always switch on the scale first and wait for the display „0.0“ (Fig. 2) before stepping on.

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