beurer MG 160

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Items included in the package:

- Massage seat cover
- Removable cover
- Power supply unit
- Car adapter
- These operating instructions

Explanation of symbols

The following symbols are used in the operating instructions and on the type plate:

WARNING Warning of risks of injury or health hazards.

CAUTION Safety information about possible damage to appliance/accessories.

Note Important information.

The device has double protective insulation and therefore complies with protection class 2.

Only use indoors.

1. Getting to know your appliance

Instrument functions

You can have your back and thighs massaged while sitting with this electrical massage device. Massages can be both relaxing and stimulating, and are often used in the treatment of stiff or tense muscles, pains and fatigue. This device massages with the aid of 6 vibration motors and offers a wide range of setting options. The adapter supplied enables the seat cover to also be used in the car. The massage seat cover has to be used when placed on a suitable seat.

The device is only for **private use** and is not intended for medical or commercial purposes.

2. Proper use

This device is solely intended for massaging the backs and legs of human beings. It cannot replace medical treatment. Do not use the massage device if any of the following warnings apply to you. Please consult your doctor if you are unsure whether or not the massage device is suitable for you.

WARNING

Use of the massage device is prohibited in the following cases:

- On children under the age of 15.
- On animals.
- On persons suffering from a herniated disk or from abnormal changes to the vertebrae.
- Not in case of cardiovascular diseases and headaches.
- Not in the case of swelling, burns, inflammation, open wounds, fresh wounds, hematomas, bleeding or other injuries to the back or legs.
- Not while you drive a car.
- Not during pregnancy.
- Never while you sleep.
- Never use the device after taking medication or consuming alcohol (reduced levels of awareness).
- Do not use the device while operating machinery.

This device is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety or they are given instructions on how to use the device.

Children should be supervised to ensure that they do not use the device as a toy.

The device has a hot surface. Persons who are sensitive to heat must be careful when using the device.

You should consult your doctor before using the massage device if:

- you suffer from a serious illness or are recovering from an operation on the upper body, buttocks or thighs.
- you are fitted with a cardiac pacemaker, implant or other aid.
- you suffer from any of the following: common cold accompanied by fever, varicose veins, thrombosis, phlebitis, jaundice, diabetes, nervous system diseases (e.g. sciatica), tuberculosis, tumors, hemorrhoids, bruises or acute inflammation.
- in case of pains of unclarified origin.

The device is only intended for the purpose described in these operating instructions. The manufacturer cannot be held liable for damage occurring as a result of inappropriate or careless use.

3. Safety information

Please read these operating instructions carefully. Failure to observe the information provided below can lead to personal injury or material damage. Keep these instructions in a safe place where they can also be read by other persons using the device. Always pass on these instructions together with the appliance if you sell it or pass it on.

WARNING

Keep children away from the packaging materials. Risk of suffocation.

WARNING: Electric shock

Just like any other electrical appliance, this massage device must be handled with due care and attention in order to prevent the risk of electric shock.

For this reason, the device must only be operated as follows:

- The massage device must only be operated with the supplied power supply unit, which must be connected to a power supply with the mains voltage indicated on the power supply unit.
- Never use the device if any damage to the appliance itself or to any of its accessories is evident.
- Never use the device in rooms in which aerosols are sprayed or oxygen is administered.
- Never use the appliance during a thunderstorm.

In the event of a defect or malfunction, switch the device off immediately and disconnect it from the power supply. Never pull on the power cable or on the appliance in order to disconnect the power supply from the mains outlet. Never hold or carry the device by the power cable. Keep the cables away from hot surfaces. Do not pinch, kink or twist the cables. Do not stick in any pins, needles or sharp objects.

Make sure that the massage device, the switch, the power supply unit and the cable do not come into contact with water or other fluids. For this reason, the device must only be operated as follows:

- Only in dry inside rooms.
- Only with dry hands.
- Never in the bath, in the shower or near a filled basin.
- Never in swimming pools, whirlpools or in the sauna.
- Never out in the open.

Never try to retrieve the device if it falls into water. Disconnect the plug immediately from the mains outlet.

Repairs

WARNING

- Repairs to electrical appliances must only be carried out by qualified persons. Incorrect repairs can lead to considerable danger for the user. In the event of repairs, please contact our customer service or an authorized dealer.
- The zip fastener on the massage device must not be opened. It is only fitted for reasons relating to the manufacturing of the device.
- The mains power cord on this device cannot be replaced. The device must be disposed of in case of damage.

WARNING: Fire hazard

There is a risk of fire if the device is used incorrectly or if it is not used according to these operating instructions.

For this reason, the massage device must only be operated as follows:

- Never operate the massage device for longer than 15 minutes (risk of overheating) and allow it to cool off for 15 minutes before using it again.
- Never leave the device unsupervised, particularly if children are around.
- Never use the device under covers, such as blankets or pillows.
- Never use the device in the vicinity of petrol or other easily flammable materials.

Handling

CAUTION

Switch the device off and disconnect the power supply unit after every use and before every cleaning.

- Avoid contact with sharp or pointed objects.
- Do not kink the device or cable.
- Do not place the device on delicate surfaces, as otherwise the zip fastener could potentially damage the surface.
- Make sure that the appliance is not too close to heat sources such as ovens, radiators or direct sunlight.

Disposal

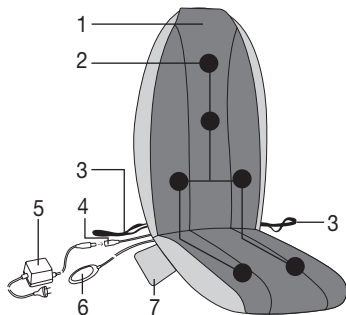
CAUTION

Follow local regulations regarding the disposal of materials. The appliance should be disposed of according to Directive 2002/96/ EC-WEEE (Waste Electrical and Electronic Equipment). In case of queries, contact the municipal authorities responsible for waste disposal in your area.


4. Appliance description

Massage seat cover (front)

- 1 Protective cover
- 2 6 massage motors
- 3 Fastening straps
- 4 Connecting coupling
- 5 Power supply unit with plug connector
- 6 Hand switch
- 7 Storage bag

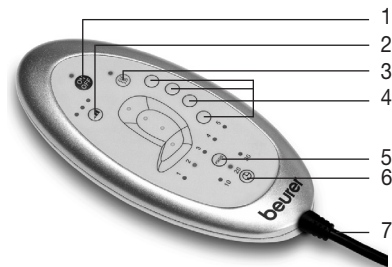Hand switch

- 1 ON/OFF button with control LED
Switch on and off
- 2 INTENSITY button with control LEDs
Select massage intensity
- 3 HEAT button with control LED
Switch the heat function on and off
- 4 MASSAGE ZONE button with control LED
Switch the various massage zones on and off
- 5 PROGRAM button with control LEDs
Select massage program
- 6 TIMER button with control LEDs
Select switch-off times
- 7 Connection cable to the device

The control LEDs indicate which functions are currently selected and which massage zones are activated.

5. Start-up

- Remove all packaging and transport locks.
Ensure that the packaging foils do not fall into children's hands. Risk of suffocation.
- Check the device, power supply unit and all cables for damage.
- Secure the protective cover on the surface of the massage seat cover using
 - the Velcro strips where the seat joins with the backrest,
 - on the reverse side under the seat surface and
 - at the top in the shoulder area.It is also possible to massage without the protective cover; this is generally found to be "harder" and more intensive.
- Position the seat cover on a suitable seat with seating surface and backrest (chair, sofa, or similar). Ensure that the seat cover is supported over its entire area and has a sufficiently high backrest.
- Use the flexible fastening straps and Velcro strips to secure the seat cover.
- Connect the power supply unit and the seat cover with the plug connector.

WARNING

- Do not subject the device to any impact and do not drop it.
 - Do not use the device if damage is evident to the device itself or any of its accessories.
 - The massage device must only be operated with the supplied power supply unit, which must be connected to a power supply with the mains voltage indicated on the power supply unit.
 - Route the cable safely so that there is no risk of anyone tripping over it.
- Connect your device to the mains by plugging the power supply unit into the mains outlet.

6. Operation

- Sit comfortably on the seat cover. Take care that you sit centered and your body is in good even contact with the seat cover.
- Switch on the massage device with the ON/OFF button. The massage intensity is preset to level 1.
- Press the PROGRAM button or at least one of the MASSAGE ZONES buttons to start the massage.
- If you wish to have a long massage with individual massage zones or all of them, select the massage zones with their respective buttons. The massage zones activated are displayed with the LEDs.
- The automatic switch-off function is activated by starting the massage and is preset to 20 minutes. You can set the automatic switch-off function to 10, 20 or 30 minutes by pressing the TIMER button. Please note that changing the timer setting makes the timer start to count again from the beginning. To avoid overheating the device, select the maximum massage duration at the beginning of the massage and do not change it again during the massage.
- If you would like a massage with one of the various programs, select the required program with the PROGRAM button. The 5 massage programs are characterized by different sequences of massage zones and massage intensities.
- Press the INTENSITY button to select of the 3 possible massage intensities.
- Press the HEAT button to switch the heat function in the lumbar region on and off.
- Do not use the massage device for longer than 30 minutes. The massage can be finished at any time by pressing the ON/OFF button.
- After the massage is finished, connect the power supply unit plug from the mains.

Note

- You can place the hand switch in the storage case during and after use.
- Always wear light clothing when receiving a massage from the device.
- Do not use the massage device for longer than 15 minutes. Otherwise, using the device for longer can over-stimulate the muscles and cause tension instead of relaxation.
- Do not use the massage device just before going to bed. The massage can also have a stimulating effect and may cause difficulties in getting off to sleep.

WARNING

The massage must be pleasant and relaxing at all times. Stop the massage or change your position or the contact pressure if you find it is painful or unpleasant.

Alternative use in the car

The massage seat cover can also be used in the car with the car adapter supplied. Plug the car adapter connector plug into the connection jack on the device. Plug the car adapter into the cigarette lighter jack or an alternative 12 V jack. (e.g. trunk). Operation proceeds as described above.

WARNING

Never have a massage with the massage seat cover while you are driving a car yourself. The massage, as well as the device setting, distracts you from the road traffic. Your attention to the road traffic must never be impaired by the device. Therefore only use this device as a passenger or if you take a break and the vehicle is stationary. Please note that if the engine is off the battery can discharge under certain circumstance. For some vehicles the 12 V jack only works when the engine is on. A massage with this seat cover can be very relaxing and regenerating while taking breaks.

7. Care and storage

CLEANING

WARNING

- Disconnect the device before cleaning it.
- Only clean the appliance in the manner specified. Fluid must never be allowed to enter into the device or accessories.
- Do not use the device again until it is completely dry.

First unplug the device and allow it to cool off. Then you can clean it with a cloth which is no more than lightly moistened. You can also use a mild detergent. The removable cover is machine-washable at 40 °C. Please follow the cleaning and care symbols on the label which is sewn into the cover.

Storage

- Keep the massage device in a cool and dry place where it is out of the reach of children. We recommend storing the device in its original packaging.
- Avoid any contact with sharp or pointed objects which could damage the device or cover.

8. In the event of problems

| Problem | Cause | Remedy |
|--|---|---|
| The device is very loud during massage. | The seat cover is not fully in contact with the base. | Ensure that the seat cover is supported over its entire area. |
| Heat function cannot be activated. | Overheating protection has been tripped. | Contact your authorized dealer or the service address. |
| Massage stopped. | The preset maximum period of operation has elapsed. | Let the device cool down for at least 15 min. before using again. Switch the device on again. |
| | The connection to the mains has been interrupted. | Check the plug connection and the contact of the mains plug with the mains supply. |
| No massage. | Appliance not connected to the mains power supply. | Make all connections and switch on the device. |
| | Device not switched on. | Switch the device on with ON/ OFF. Press PROGRAM or MASSAGE ZONES and select the settings. |
| | Overheating protection tripped, device is damaged. | Contact your authorized dealer or the service address. |

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