beurer

HKM 60

- Ⓓ **Akku-Heizkissen**
Gebrauchsanleitung
- ⒼⒷ **Battery-Operated Heating Pad**
Instruction for Use
- Ⓕ **Coussin chauffant à piles**
Mode d'emploi
- Ⓔ **Almohada eléctrica con acumulador**
Instrucciones para el uso
- Ⓘ **Termocuscino a batteria**
Istruzioni per l'uso
- Ⓓ **Akülü ısı yastığı**
Kullanma Talimatı
- Ⓔ **Акумуляторная электрогрелка**
Инструкция по применению
- Ⓓ **Poduszka elektryczna z akumulatorem**
Instrukcja obsługi

Zeichenerklärung Etikett • Label legend • Légende de l'étiquette • Explicación de los símbolos de la etiqueta • Spiegazione dei simboli sull'etichetta • Şekillerin Açıklamaları Etiketi • Объяснение условных знаков на этикетке • objaśnienie oznaczeń etykiety

Hinweise lesen!
Read instructions!
Lire les instructions!
Leer las instrucciones!

Leggere le avvertenze!
Açıklamaları Okuyunuz!
Читайте указания по применению!
Czytać wskazówki!

Nicht gefaltet oder zusammengeschoben
gebrauchen!
Do not use when folded or creased!
Ne pas utiliser plié ou comprimé!
No utilizar estando plegado o arrugado!
Non utilizzare ripiegato né accartocciato!

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ENGLISH

Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

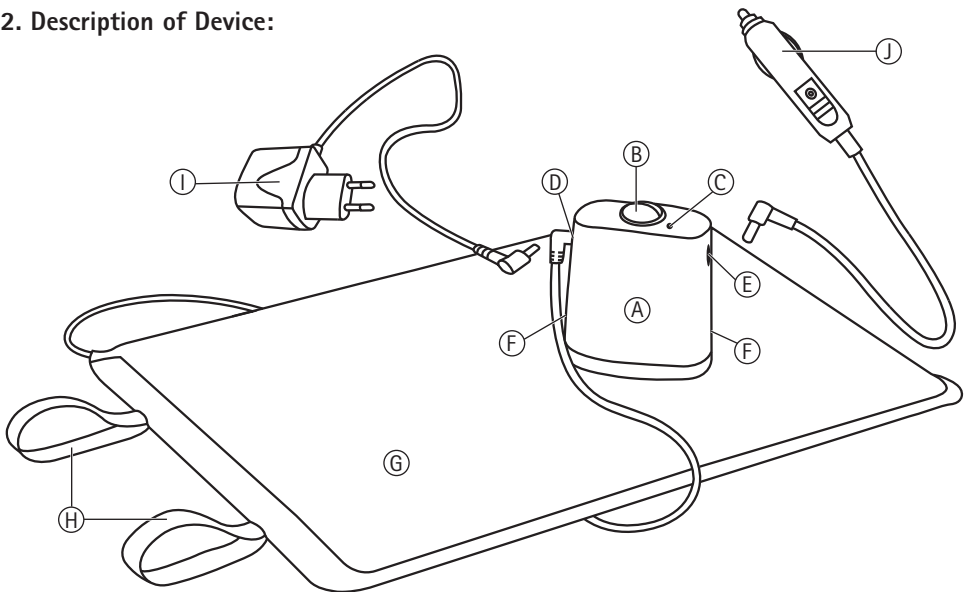
Best regards,
Your Beurer Team

1. Important information – retain for later use

- Read instructions carefully.
- This heated pad is not intended for use in hospitals.
- Do not use with helpless persons or persons insensitive to heat (e.g. diabetics, persons with illness-related skin conditions or regions of skin scarring in the application area, following consumption of pain-relief medication or alcohol).
- Do not use to warm animals.
- Do not apply the heating pad to parts of the body that are inflamed, injured or swollen. In case of doubt you must firstly obtain medical advice.
- Caution! Do not, under any circumstances, fall asleep while the heated pad is in use.
- Excessively long use can cause the skin to be burned.

- The electrical and magnetic fields emitted by this electrical product may, under certain circumstances, interfere with the function of your cardiac pacemaker. However, they are far below the permissible limits: electrical field strength: max. 5000 V/m, magnetic field strength: max. 80 A/m, magnetic flux density: max. 0.1 milli tesla). Therefore, please consult your doctor and the manufacturer of your pacemaker before using this product.
- The heating pad must not be used unattended.
- The heating pad must never be used folded or rolled up.
- The battery-operated heating pad must never be used in conjunction with other heating devices, e.g. car seat heating pads or heating underblankets.
- The heating pad should not be sharply bent.
- Do not pull on, twist or sharply bend the supply cords.
- Do not stick in any pins, needles or sharp objects.
- The battery pack and connections must be protected against moisture, impacts or direct sunlight.
- The heating pad must be checked frequently for signs of wear or damage. If such signs are present, or if the appliance has been used improperly, it must be taken to either the manufacturer or the dealer before further use.
- Repairs to electrical appliances may only be carried out by specially trained persons. Incorrect repairs can result in considerable danger to the user.
The battery in the battery pack must not be disassembled, thrown into fire or short-circuited.
- Only use the heating pad with the original battery pack (type AP1) supplied.
- Only use the original battery charger (type P-001B-B1010) to charge the battery pack. The battery charger is only designed for indoor use and must not be exposed to any moisture.

2. Description of Device:

- | | |
|---|--|
| (A) Battery pack | (G) Heating pad |
| (B) Switch for temperature settings | (H) Rubber bands to secure the rolled up heating pad |
| (C) Function display for the battery pack | (I) Battery charger |
| (D) Connection for battery charger or heating pad | (J) Car adapter |
| (E) Connection for car adapter | |
| (F) Lock for battery pack housing | |

3. Operation

With this heating pad you can use directed warmth and you are no longer reliant on an electrical outlet. For every conceivable outdoor activity, e.g. in the sports arena, on camping trips or fishing, this battery-operated heating pad offers you more than 2 hours (depending on the temperature setting) of pleasant warmth.

The practical storage bag enables easy transportation. Rolled up and secured with the rubber bands, the heating pad can be transported in it with ease.

Its body-hugging and practical shape means the battery-operated heating pad is suitable for use both as a seat cushion as well as under clothing.

3.1. Charging the battery pack

- **Important:** Prior to initial use or after prolonged periods without use, the battery pack must be charged for at least 14 hours.
- Make sure that the car adapter is separated from the battery pack for this purpose.
- Then plug the charger into the electrical outlet with a voltage corresponding to the specifications on the charger. The function display on the charger is illuminated green.
- Select the switch position "0" (Off) on the battery pack and connect the plug with the connector on the battery pack. The function display on the charger is illuminated red. The function display on the charger (green/red) signals that the charger is functioning properly but does not show the actual charge status.
- Only charge the battery pack in a dry environment and at temperatures between 10 °C and 35 °C.
- Please note that the full capacity of the battery pack is only reached after several charging and discharging cycles.

3.2. Operating the battery-operated heating pad with the battery pack

- Remove the charger plug from the battery pack.
- Connect the heating pad plug with the battery pack.
- The red heating pad surface is intended for use on the body.
- Select the required temperature setting with the switch. The function display of the battery pack flashes briefly and then remains continuously illuminated.

Temperature settings:

- 0 = Off
- I = Minimum heat (indicated by 1x flash)
- II = Moderate heat (indicated by 2x flashes)
- III = Maximum heat (indicated by 3x flashes)

We recommend you select the highest temperature setting for fast warming. You can switch back to a lower temperature setting later as required.

When the battery is discharged the function display of the battery pack lights up. The heating pad then no longer provides warmth. Charge the battery pack as described in 3.1.

The function display of the battery pack lights up when the battery is charged even if the heating pad is not connected.

Please note that the battery pack should be protected against extreme cold (under clothing if applicable) to maintain its full efficiency.

3.3. Using the battery-operated heating pad in the car (in the 12 V cigarette lighter socket)

- Connect the heating pad plug with the battery pack.
- Connect the car adapter to the battery pack.
- Connect the car adapter with the cigarette lighter socket.
- Select the temperature setting (see 3.2).
- **Please note that the battery-operated heating pad must not be used in conjunction with any car seat heating device present.**
- If the battery-operated heating pad is not used, it should be switched off (switch setting "0") and the car adapter disconnected.
- The battery is not charged during car operation!
- To protect your car battery against discharging, only in car operation there is an automatic switch-off after approx. 3 hours. After the 3 hours the function display of the battery pack starts to flash.

3.4. Replacing the battery from the battery pack

If the heating pad only warms for a brief period with a fully charged battery pack, we recommend replacing the battery in the battery pack with the original spare part (see 3.5).

To replace the battery, open the battery pack housing by pressing the two locking surfaces together and pulling out the housing lid from the housing at the same time. Then remove the plug connector from the battery.

3.5. Spare parts

You can obtain the following spare parts directly from Beurer Customer Service:

- Battery pack (Item No. 206.081)
- Battery (Item No. 162.271)
- Battery charger (Item No. 546.003)
- Car adapter (Item No. 108.811)

4. Cleaning and Care

The battery-operated heating pad has a water repellent surface. A cloth or moist sponge, together with some liquid detergent for fine fabrics if necessary, can be used to remove smaller stains. Never use any cleaning agents containing solvents!

5. Storage

If you do not use the heating pad for longer periods, we recommend storing it in the original packaging in a dry place without weighting it down.

6. Disposal

Please dispose of the appliance in accordance with the Directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE). In case of queries, please contact the municipal authorities responsible for waste disposal in your area.

The battery must firstly be removed (see 3.4) and also properly disposed of.

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