beurer

HK 55 Easyfix

- | | |
|--|--|
| [D] Heizkissen
Gebrauchsanleitung | [I] Cuscino riscaldante
Istruzioni per l'uso |
| [GB] Heating pad
Instruction for Use | [TR] Isıtmalı yastık
Kullanma Talimatı |
| [F] Coussin chauffant
Mode d'emploi | [RUS] Электрическая грелка с поясом
Инструкция по применению |
| [E] Almohadas eléctricas
Instrucciones para el uso | [PL] Poduszka elektryczna
Instrukcja obsługi |

**Inhaltsverzeichnis • Table of content • Sommaire • Índice • Sommario
• İçindekiler • Оглавление • Spis treści**

D	Deutsch	Seite	4-7
GB	English	Page	7-10
F	Français	Page	10-13
E	Español	Página	13-16
I	Italiano	Pagina	16-19
TR	Türkçe	Sayfa	20-22
RUS	РУССКИЙ	Стр	23-26
PL	Polski	Strona	27-29

**Zeichenerklärung Etikett • Explanation of symbols • Légende des symboles
• Explicación de los símbolos • Spiegazione dei simboli Etichetta • Etiket işaret
tarifi • Объяснение условных знаков на этикетке • Objasnienie oznaczeń etykiet**

Hinweise lesen!
Please read the instructions!
Lire les remarques!
¡Leer las indicaciones!

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ENGLISH

Dear Customer,

We are happy that you have decided on a product from our range. Our name stands for high-quality and thoroughly tested products from the areas of warmth, blood pressure, body temperature, weight, gentle therapy, massage and air.

Please read these instructions for use carefully and follow the directions.

Best regards,
The Beurer Team

Items included in the package: 1 heating pad, 1 switch, these operating instructions

1. Important safety information – read carefully and keep for later use

Failure to observe the information provided below can lead to personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information not only serves to protect your health and the health of others, it also protects the product from damage.

Therefore, observe this safety information and pass on these instructions together with the product.

- Only use this heating pad for the purpose described in these instructions.
- This heating pad is not intended for use in hospitals.

- This heating pad is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety.
- This heating pad should not be used with persons who are helpless, small children or persons insensitive to heat (e.g. diabetics, persons with skin changes related to illness or persons with scarred skin in the area where the pad is to be applied, after taking any pain-relieving medication or alcohol).
- Excessively long use can cause the skin to be burned.
- Caution! Never fall asleep while the heating pad is switched on.
- Do not apply the heating pad to parts of the body which are inflamed, injured or swollen. If in doubt, seek medical advice before use.
- The electric and magnetic fields emitted by this heating pad can interfere with the operation of pacemakers. However, they are far below the permissible limits: electric field strength: 5,000 V/m max., magnetic field strength: 80 A/m max., magnetic flux density: 0.1 millitesla max. Therefore, please consult your doctor and the manufacturer of your pacemaker before using this heating pad.
- Heating pad
 - only connect to the voltage specified on the appliance,
 - never operate without supervision,
 - never switch on when folded or compressed,
 - never wedge between objects,
 - never make any sharp creases,
 - never use with animals,
 - never use when damp,
 - only use in conjunction with switch types specified on the heating pad.
- When in use, the electronic components in the switch of the heating pad make the switch slightly warm. The switch should therefore never be covered or placed on the heating pad when it is being operated.
- Do not pull, twist or make any sharp kinks in the cables.
- Do not stick pins or pointed objects into this heating pad.
- This appliance may not be used by children unless the switch has been preset by a parent or a supervisory person, or the child has been sufficiently instructed in how to use this heating pad safely.
- Children should be supervised to ensure that they do not use the heating pad as a toy.
- This heating pad must be checked frequently for signs of wear or damage. If such signs are present, or if the cables are damaged, or if this heating pad has been used incorrectly, it must be taken to either the manufacturer or the dealer before further use.
- Repairs may only be carried out by specially trained personnel or at an authorised repair workshop, as special tools are required. Incorrect repairs may cause considerable danger to the user.
- Should you still have questions on using our appliances, please contact our customer service.

2. Proper Use

This heating pad is designed for warming the human body. It allows you to apply targeted warmth. Heat promotes the circulation and relaxes the muscles. This heating pad is not intended for use in hospitals or for commercial applications. In particular, it may not be used on infants, small children, persons insensitive to heat, helpless persons or animals.

This heating pad is made of breathable velour and is very adaptable and agreeable to the skin. The detachable switch allows the pad to be cleaned in the washing machine (see cleaning instructions).

When used for the first time, the heating pad may start to smell of plastic but this will soon go away. This heating pad can be used all purposes, for example, on the stomach, back, joints or neck. If used for the neck area, the heating pad can be rolled up.

The pull-out Velcro fastenings make it very easy to fasten to the body. The elastic ensures that the heat is always positioned optimally.

3. Operation

3.1 Safety

This heating pad has a SAFETY SYSTEM. This electronic sensor technology prevents the entire surface of the heating pad from overheating by automatically switching off in the event of a fault. If the safety system switches off the heating pad automatically as a result of a fault, the indicator lamp is no longer illuminated even when the heating pad is switched on. Please note that, after a fault has occurred, the heating pad can no longer be used for safety reasons and must be sent to the service address indicated.

3.2 Selecting temperature

The fastest way to heat up the heating pad is to first set the switch to the highest setting. A lower setting can be selected later if required.

After switching on the heating pad, the setting starts flashing briefly, while the heating pad conducts a self-test. Then the setting remains illuminated continuously.

We recommend the lowest temperature setting for extended use of the heating pad.

3.3 Temperature settings

- Setting: 0 Off
- 1 Minimum heat
 - 2 Medium heat
 - 3 Maximum heat

3.4 Fast heating

These heating pads have a fast heating function which allows them to heat up quickly within the first 10 minutes.

3.5 Automatic switch-off function

These heating pads have an automatic switchoff function which stops the heat supply approx. 90 minutes after switching on the pad. The illuminated function display then starts to flash. To operate the heating pad again, you must first set the switch to zero. After about 5 seconds, it can be switched on again.

If the heating pad is no longer used after the timed switch-off, it should be switched off (setting "0") or else unplugged from the mains.

4. Cleaning and Care

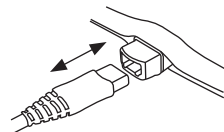
Before cleaning the pad, always unplug it from the mains. Smaller stains can be removed using a cloth or moist sponge together with a little liquid detergent for delicate fabrics if necessary. Do not use any cleaning agents containing solvents on the appliance.

Please note that the heating pad should not be dry-cleaned, wrung out, machine-dried, pressed or ironed.

Before washing, first remove the plug-in connection together with the switch from the heating pad. Set the washing machine to a gentle washing cycle at 40°C.

For ecological reasons, we recommend that the heating pad is only washed together with other textiles. Use a washing powder or liquid for delicate fabrics and dose it according to the manufacturers instructions.

Please note that excessive washing will wear out the heating pad. For this reason, the heating pad should only be machine-washed a maximum of 10 times throughout its entire life.

When drying the heating pad, do not use clothes pegs or anything similar, and never switch the heating pad on to dry it!

Do not reconnect the switch to the heating pad until the plug-in connection and the heating pad are completely dry.

5. Storage

If you are not going to use the heating pad for some time, we recommend storing it in its original packaging in a dry place without weighing it down.

Allow the heating pad to cool down beforehand.

6. Disposal

Please dispose of the blanket in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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