



UNINSTALL DRIVER:

Uninstall driver on Windows XP:

If you use Windows XP, you can uninstall the driver with the program "DriverUninstallXP.exe":

1. Save this program on your PC.
2. Start the deinstallation by double-click.
3. Click the button "Uninstall driver" and confirm the process with "Yes". Now the driver is uninstalled.

If Windows 2000 is installed, please follow the steps in the next paragraph.

Manual driver removal on Windows 2000 and XP:

1. Connect the heart rate monitor to the PC.
2. Click "My Computer" on your desktop with the right mouse button and choose "Manage".
3. Click "Device Manager" in the left area of the window.
4. Then open the area "USB Controller" by double click in the right area of the window.
5. If a previous installation has not been completed successfully the point "USB Bridge" will be marked with a yellow exclamation mark. Alternatively the USB Bridge can be found in "Other devices".
6. Click "USB Bridge" with the right mouse button. Then choose "Uninstall" on the right side.
7. Finally confirm the process with "Ok". Now the driver is uninstalled.